KULLANIM AMACI

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INDICATION

Across LISS is used for preparing erythrocyte suspensions, dilution of serum/plasma used in gel techniques.

PRINCIPLE

LISS (Low Ionic Strength Solution) significantly increases the antigen-antibody interaction by lowering the ion density in the environment. The use of this solution facilitates the binding of the antibody to red blood cells, reducing the time to form the antigen-antibody complex.

CONTENT

- Contains buffer solution with low ionic strength and preservatives.
- Reagents are ready for use.

STORAGE CONDITIONS

- Store Across LISS Solutions at 2-25°C.
- Do not freeze.
- Do not store at high temperature.
- If stored properly, it is stable until expiry date provided that contamination is prevented.

MATERIAL REQUIRED BUT NOT PROVIDED IN THE KIT

- Across Gel cards
- Follow the Instructions for use of the Across Gel cards to be used.

COLLECTION AND PREPARATION OF SAMPLES**Samples**

Blood samples with anticoagulant (containing plasma) can be used. Do not use haemolysed, cloudy, contaminated blood samples or with clot presence.

To detect antigens on erythrocyte surface, work with:

- Fresh blood samples collected in tubes containing anticoagulants like EDTA, citrate etc. can be used.
- Blood samples stored at 2-8°C up to 48 hours after extraction can be used.
- Blood samples stored at 2-8°C, collected from bags containing ACD, CPD, CPDA and SAG-Mannitol which are not beyond the expiry date specified on the label can be used.

Sample Preparation:**a) Preparing erythrocyte (red blood cell) suspension****(To detect ABO/Rh- Kell system antigens)**

To prepare 5% erythrocyte suspension, add 25 µl erythrocyte to 0,5 ml Across LISS.

b) Preparing erythrocyte (red blood cell) suspension (To screen unexpected antibodies such as Direct Coombs, Cross-Match)

To prepare 1% erythrocyte suspension, add 10 µl erythrocyte to 1 ml Across LISS.

TEST PROCEDURE

It can be worked with Across LISS manual equipment or Across System Auto semi-automatic and automatic instruments. For Across System Auto semi-automatic and automatic instruments, see the instrument's user manual. The working principles of Across LISS solution in manual equipment and Across Auto System semi-automatic and automatic instruments are equivalent.

WARNINGS AND PRECAUTIONS

- The product must be used by a qualified staff member.
- Preparing the red blood cell suspension in a concentration other than the recommended concentration may give a false negative or a false positive test result.
- Using a solution other than Across LISS in preparing red blood cell suspensions may change the reaction.
- Adding a volume other than that stated in the methods in the Across Gel cards may change the reaction.
- Do not use a solution with past expiry date.
- Do not use the product in the presence of microbial or fungal contamination.
- Do not use the product if the package is damaged.
- The product is recommended to be disposed of into a special container used for biological waste after use.

PACKAGING**Across LISS**

Ref No : 830100 ---- 100 ml
830500 ---- 500 ml

Document No: TK2703

This document is available in several languages. Translations have been made from the IFU in Turkish. In case of doubt or discrepancy, statement in master Turkish document shall be taken into consideration.